



Agree to Instructions for Use

- 1 Scroll through the document to read then tap **Done**

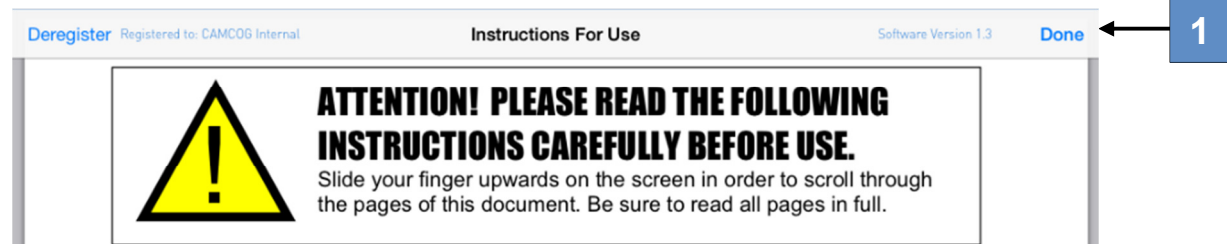


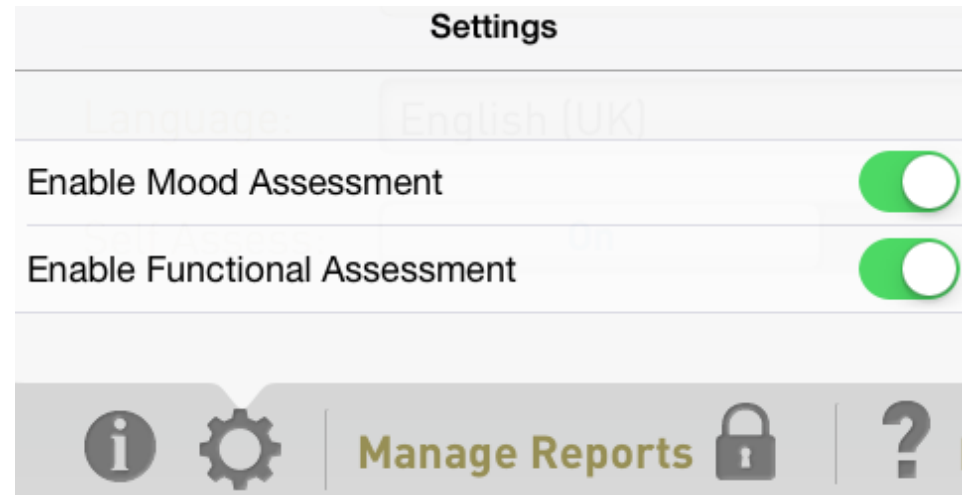
Adding Patient Details

- 2 Tap in the patient I.D. field and enter the patient identifier
- 3 Enter the patient's DOB
- 4 Tap either 'male' or 'female' box depending on the sex of the patient
- 5 Select the patient's level of education from the drop-down box
- 6 Select the language required. This applies to the PAL memory test, GDS and ADL, but the report will be given in English
- 7 Tap 'on' or 'off' for the self-assessment (this is one question asking the patient to rate their own memory)
- 8 Tap **Run Test**

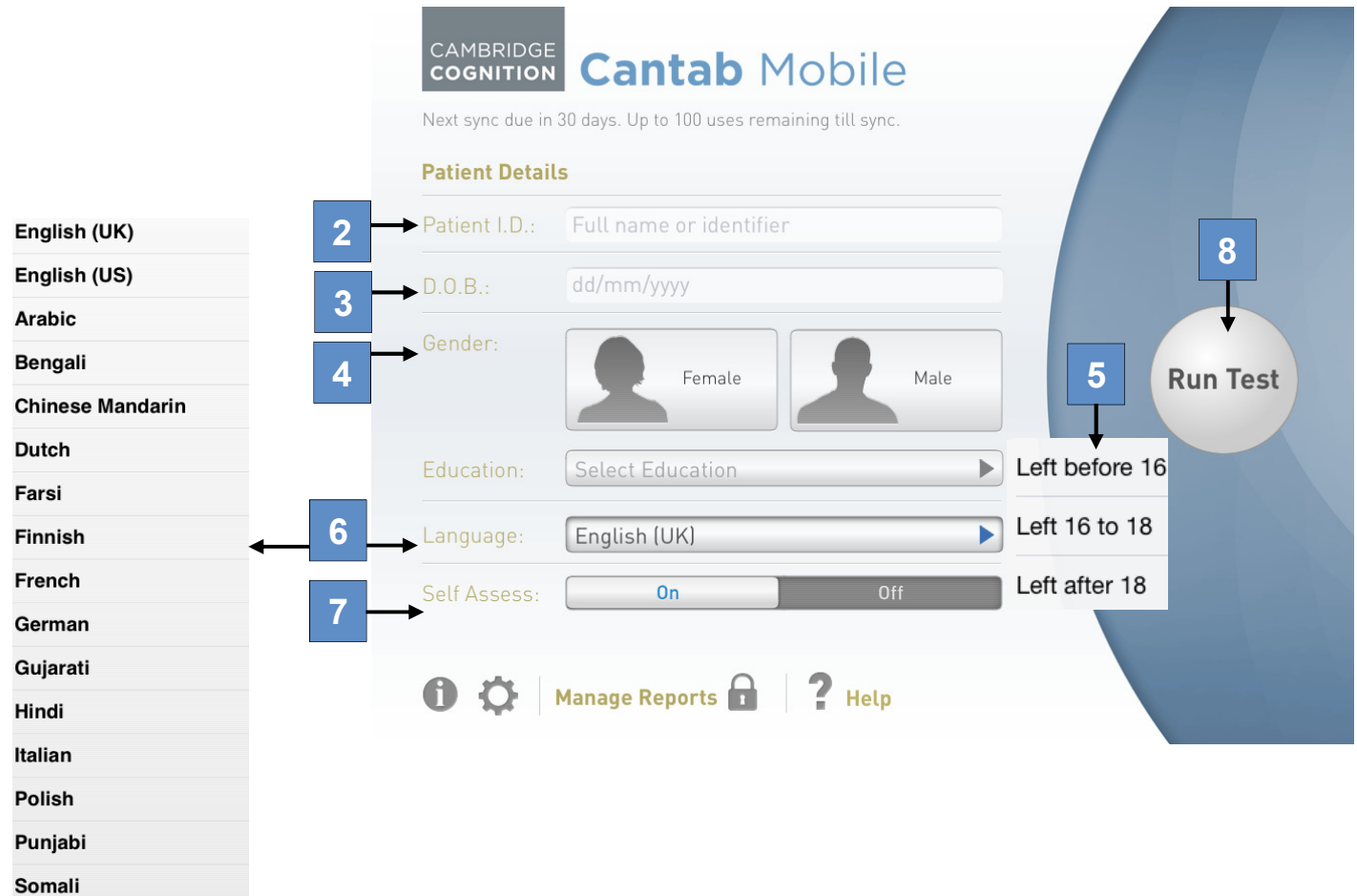



Settings


- 9 To open 'settings', tap 
- 10 To activate GDS, ensure 'Enable Mood Assessment' is ON (recommended)
- 11 To activate ADL, ensure 'Enable Functional Assessment' is ON (recommended)




- 10
- 11

Support

- 12 To open 'support', tap  **Help**
- 13 Scroll through the Help Videos and tap **Play video** as required
- 14 For any support enquiries, tap  **Support**
- 15 This will open an email template. Add details of your query and click **Send** (NB: this requires a Wi-Fi connection)
- 16 This will automatically send to support@cantabmobile.com
We aim to respond to all enquiries within 24 hours

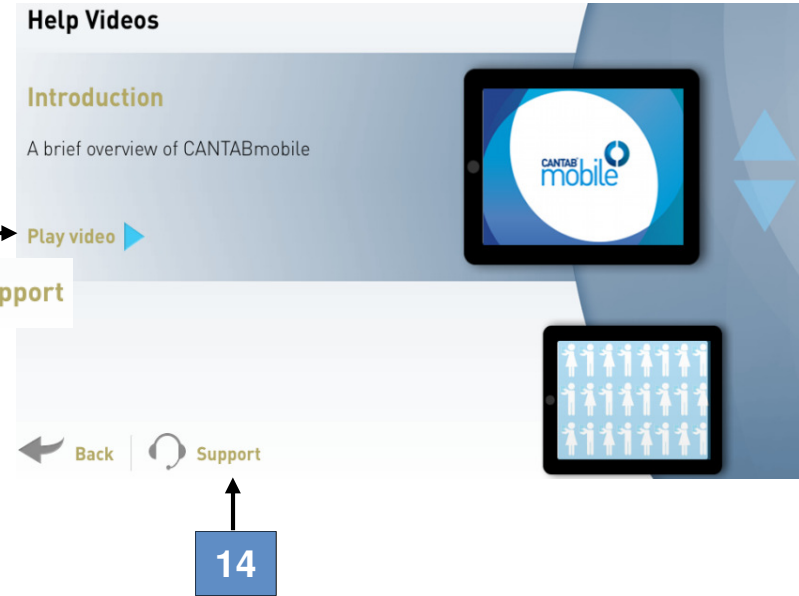

- 13
- 14
- 14

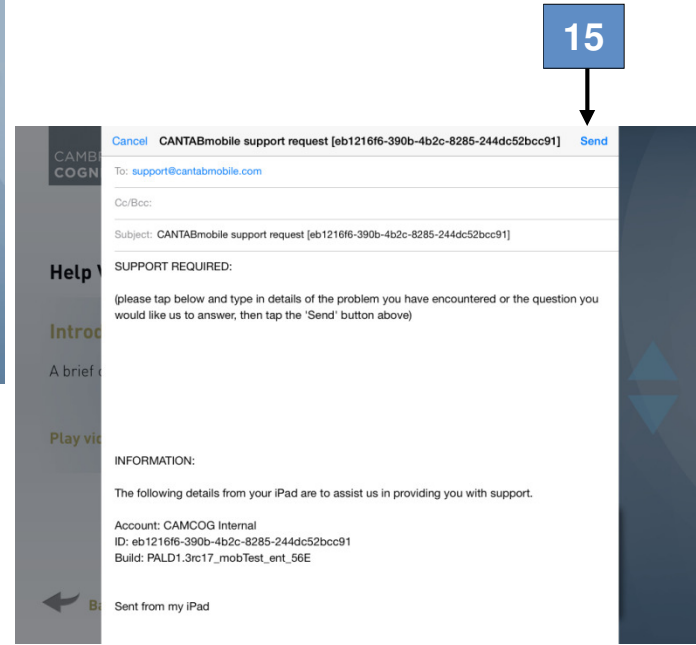

- 15